### PROTECTIVE SPRAY

Cat. No. 47007

- Recommended following dehorning.
- **Relieves** pain
- Produces a **cooling action** on the wound
- **Desinfects**
- Protects: It has a «second skin» effect that allows the wound to breathe
- **Repellent** (especially for flies)
- Water impermeable, does not run and does not wipe off
- For cattle, goats, sheep, pigs and horses.

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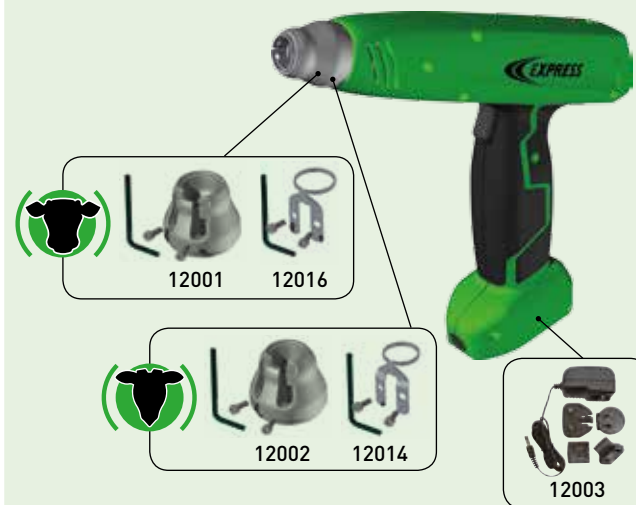
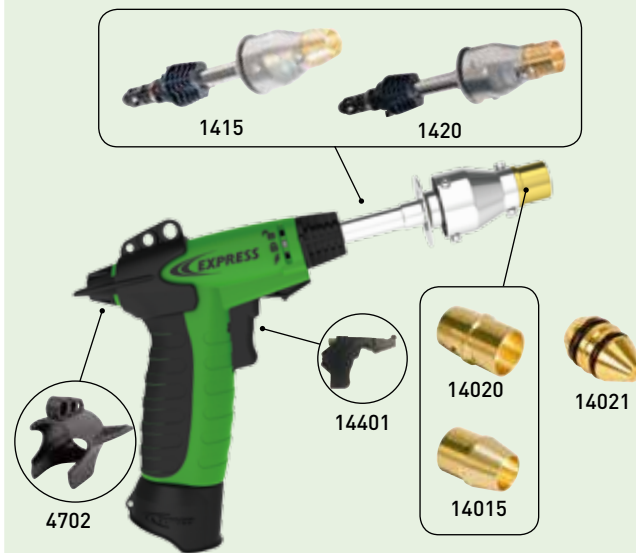
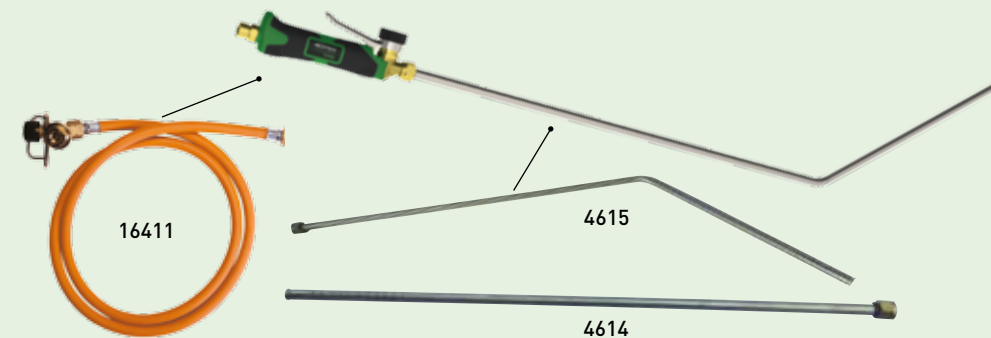
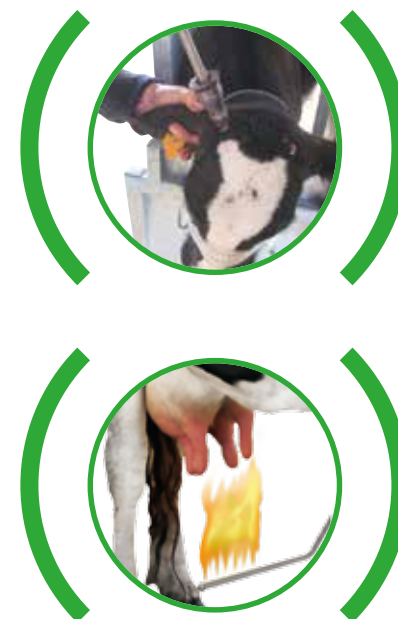
# PORTABLE HAIR REMOVER

## DEPILADORA/CHAMUSCADORA DE UBRES AUTÓNOMA EPILATORE AUTONOMO

Cleaning udder to improve hygiene before milking avoids **germs, bacteria, mastitis and lost time.**

For more than 15 years, Express has provided the solution: **TERRA**, the portable thermal hair remover!

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**15655**\*

**TERRA**

- EASY** • Set up in less than **2 minutes**
- FAST** • Hair removal in **less than 4 seconds** per cow
- SAFE** • Easy and safe hair removal due to the **long curved burner**
- PAINLESS** • Thanks to the **"cool flame"**, the animal feels only a sensation of warmth
- HANDY** • Cartridge weight 340 g with duration of **2 hours**
- EFFECTIVE** • Reduces the spread of germs and risk of mastitis
  - Faster cleaning of the udder before milking
  - Easier teat identification for automated milking
  - Improved milk quality
- ECONOMICAL** • Avoids multiple processing of milk. Compatible with gas cylinder/cartridge

- ▶ Handle + angled burner: ± 500 g
- ▶ Burner length: 1000 mm
- ▶ 15655: Handle + angled burner + 1.50 m hose + 1 butane/propane cartridge (Cat. No. 556) (Gas cartridge not included).

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▶ Keep the flame at a distance of approximately 10 to 20 cm while drawing an "eight" for 2 to 4 seconds.

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AGE < 29 DAYS  
EDAD < 29 DÍAS  
ETÀ < 29 GIORNI

AGE < 15 DAYS  
EDAD < 15 DÍAS  
ETÀ < 15 GIORNI

To avoid risks of injury to animals and farmers, Express provides gas solutions for dehorning young calves and goats:

- **FAST**  
± 5 seconds per horn bud (no charging time required)
- **RELIABLE**
- **SAFE**  
in damp conditions

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## WHEN TO DEHORN

It is recommended to perform dehorning as soon as possible, before the age of 4 weeks for calves and before 2 weeks for kids.

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## HOW TO DEHORN

### BEFORE

- ▶ Immobilize the animal in a cattle crush.
- ▶ Shave around the horn bud before cauterizing in order to:
  - clearly see the location of the future horn bud,
  - avoid getting hair in the wound, which can be a source of infections,
  - clearly see how well the cauterization has been performed.

### DURING

- ▶ Apply the tip of the dehorner on the horn bud to cauterize the blood vessels and stop its growth.
- ▶ Do not tear off the horn bud in order to limit the potential infection area. The horn bud will dry and fall off by itself.

### AFTER

- ▶ Carefully look at the cauterization ring.

- ▶ A 5 mm-deep groove should be visible around the horn bud. There should be nothing connecting the skin and the horn bud.
- ▶ Apply the antiseptic and repellent spray.

### It is recommended to:

Either inject a tranquilizer or, preferably, apply local anaesthetic to the cornual nerve, in order to ensure the comfort of the animal and farmer.

Use a long-acting anti-inflammatory drug to limit harmful consequences of dehorning: refusal to use headlock panels, loss of appetite, isolation.

Avoid dehorning while another treatment (vaccine, etc.) is being administered or a new diet is being given.

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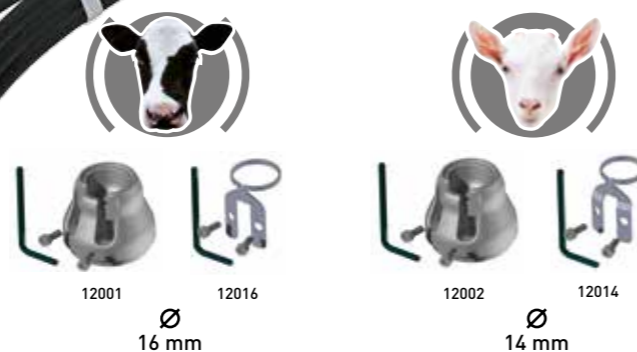
12002  
12014  
Ø  
14 mm

## CONTROL YOUR TIME

- ▶ Always ready to use: instantly heats and cools down (Temp. 700 °C / 1300 °F).
- ▶ Automatically stops when the 10-second dehorning cycle is over.

### COSMOS BATTERY version

- Cat. No. 124: With Ø 16 mm cattle tip
- Cat. No. 125: With Ø 14 mm goat tip
- ▶ 1 battery charger supplied.
- ▶ ± 760 g

### COSMOS MAINS version

- Cat. No. 126: With Ø 16 mm cattle tip
- Cat. No. 127: With Ø 14 mm goat tip
- ▶ Convenient to work with: 6 m cable.
- ▶ ± 560 g

### Spare parts

- ▶ Tip formed of a ceramic element and heating wire:
  - 16 mm internal Ø cattle tip
  - 14 mm internal Ø goat tip
- ▶ Charger Cat. No. 12003

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11125

## THE BENCHMARK

- ▶ Compact, all-metal
- ▶ No cable/Portable: butane/propane/propene cartridge Cat. No. 446
- ▶ Built-in Piezo ignition
- ▶ Temp.:  $\pm 600\text{ }^{\circ}\text{C}$  /  $\pm 1112\text{ }^{\circ}\text{F}$
- ▶ Max temp. reached in less than 5 minutes
- ▶ Support stand
- ▶ Tip internal  $\varnothing$  19 mm Cat. No. 11120
- ▶ Additional tips to best adapt to various calf breeds:
  - Tip internal  $\varnothing$  15 mm Cat. No. 11115
  - Tip internal  $\varnothing$  26 mm Cat. No. 11125
- ▶  $\pm 770$  g

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14015

## THE PISTOL GRIP

- ▶ Lightweight: ± 470 g
- ▶ No cable/Portable: butane/propane/propene cartridge Cat. No. 446
- ▶ Built-in Piezo ignition
- ▶ Temp.: ± 700 °C / ± 1292 °F
- ▶ Max temp. reached in less than 3 minutes
- ▶ Safety:
  - Lock position
  - Operation position
- ▶ Easy to handle:
  - Non-slip dual-material handle
  - Pistol shape for steady handling
- ▶ Support stand
- ▶ Reversible tip internal ø 17 and 19 mm Cat. No. 14020, to best adapt to various calf breeds
- ▶ Additional tip internal ø 15 mm Cat. No. 14015
- ▶ Self-maintenance: change the trigger/ Piezo-ignition and gas nozzle without tools

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**DAOS** 114/1 114/2

### THE CLASSIC

- ▶ Lightweight: ± 300 g
- ▶ Compatible with gas cylinder/cartridge
  - 114/1: Supplied with butane/propane cartridge (Cat. No. 556) and 1.50 m hose
  - 114/2: Supplied with 4.75 m hose and 2-bar regulator
- ▶ Temp.: ± 700 °C / ± 1292 °F
- ▶ Max temp. reached in less than 3 minutes
- ▶ Support stand
- ▶ Reversible tip internal ø 17 and 19 mm Cat. No. 14020, to best adapt to various calf breeds
- ▶ Additional tip internal ø 15 mm Cat. No. 14015

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## RESTRAINT CAGE FOR CALVES

Cat. N°. 47009

- Our cage is designed for dehorning young calves aged 0 to 4 weeks, in order to ensure effective restraint.
- It **can be transported** in a van **without being dismantled**.
- Fully galvanised to ensure a **long lifetime**.
- 1 tray to place the spray, dehorner and clipper.
- **Adjustable restraint ring**: no risk of strangling.
- Front door with 180° opening.
- Large inflatable wheel for easy transport.
- Embossed non-slip floor mat.
- Overall dimensions: 1350 x 1350 x 340 mm.
- Weight: 66.5 kg.

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## DEHORNING RESTRAINT SYSTEM SPECIALLY DESIGNED FOR KIDS

Cat. No. 47008

- Can be **adapted** to fit any type of gate.
- Very easy to install.
- **Convenient to work with**.
- Optimal head support.
- **Self-adjustable** according to the kid's head.
- Left-right **reversible**.
- Body fully supported.
- **Ergonomic**.
- Includes a tray to place equipment (clipper, dehorner, spray).
- Weight: 4.5 kg.
- Dimensions of assembled cage: L 51 x W 44 x H 31 cm.

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## CLIPPER KIT FOR DEHORNING

Cat. No. 47017

- Includes :
  - 1 clipper
  - 1 brush
  - 1 comb
  - 1 bottle of oil
  - 1 support
- 1 comb size designed to properly clear the area.
- **No cable**.
- Long battery life. (100 mins).

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